

**Multi-Gyn  
FloraPlus  
Australia**

**karo**  
pharma

Reason for change:  
Correction round: 3  
Status:

Address change Bioclin / MOA 2QR  
Date: 20-07-2021

**IN PROGRESS / PRINT FILE**


KARO code: NK1801\_6  
Previous code: NK1801\_5  
CMO code: XXXX  
Component type: Leaflet  
Language: English  
Pharma code: -

Dimensions (HxW): 230 x 150 mm  
Machine drawing ID: XXXXXX  
Material: ?


Print colours:



Onlay:

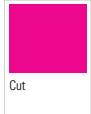

**ATTENTION:**

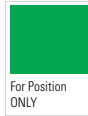

If this supporting colour swatch is present it means **the artwork is still in progress** and therefore its content and technical specs are not ready for reproduction.



READ THIS INFORMATION LEAFLET CAREFULLY BEFORE USING MULTI-GYN FLORAPLUS AND KEEP IT FOR REFERENCE

## MULTI-GYN FLORAPLUS FOR VAGINAL YEAST PROBLEMS

- + For the direct relief of itch, irritations and crumbly, white discharge
- + Contains prebiotics
- + Has the ideal pH for a healthy vagina
- + Contains plant derived ingredients

## INDICATIONS

Multi-Gyn FloraPlus is suitable for women suffering from unpleasant discomforts related to vaginal yeast and related symptoms like itch, irritations and crumbly, white discharge.

Does not treat or prevent Sexually Transmitted Diseases.

## HOW MULTI-GYN FLORAPLUS WORKS

Multi-Gyn FloraPlus contains 2QR-complex in combination with prebiotics. The 2QR-complex is a natural plant-derived substance that has the unique ability to help prevent harmful bacteria and yeast from contacting the vaginal tissue and mucosa. The prebiotics support the "good" lactobacilli to restore an optimal vaginal flora. This unique combination makes Multi-Gyn FloraPlus ideal for vaginal yeast problems and restoring vaginal health.

## DIRECTIONS FOR USE

For vaginal yeast problems, use 1 single dose tube just before you go to sleep for 5 consecutive days.

If your complaints persist or recur, you should contact your health professional. Multi-Gyn FloraPlus should not be used for longer than 30 consecutive days. This allows the vaginal flora to regulate itself without support.

## PLEASE NOTE


You may experience a burning and/or tingling sensation after application. This is normal and the sensation will pass quickly.

## MULTI-GYN FLORAPLUS IS BASED ON NATURAL INGREDIENTS

Multi-Gyn FloraPlus does not contain hormones, ingredients of animal origin or perfume.


## HOW TO USE MULTI-GYN FLORAPLUS

Multi-Gyn FloraPlus is a prebiotic vaginal gel which is applied directly into the vagina. Each tube is a single dose application. The drawings below show how to use the single dose tube. Dispose of the tube after use.

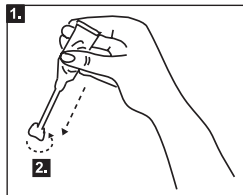


1. Shake the tube contents against the tip of the tube.


2. Break off the tip of the tube by twisting it.

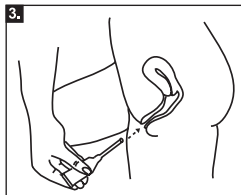

3. Gently insert the thin end of the applicator (of the tube) into the vagina.

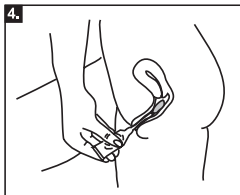

4. Gently squeeze the tube to apply the gel into the vagina.

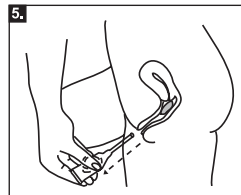

5. Continue squeezing the tube while removing it.

**Note:** The tube is designed to apply the recommended amount of gel. When used as directed, a small amount of gel will remain in the tube.

### Consumer information:

(AU) 1800 506 750  
(NZ) 0800 555 895

## MAIN INGREDIENTS

Galactoarabinan Polyglucuronic Acid Crosspolymer\*, Prebiotics (Lactitol and FOS).

\*2QR-complex: patented ingredient purified from Aloe Barbadosis.

## ALLERGY INFORMATION

Allergies to the ingredients of Multi-Gyn FloraPlus are very rare. If you are sensitive to any of the ingredients, test a small amount on the inside of your arm. If an itchy red spot or patch appears, we advise you not to use Multi-Gyn FloraPlus.

## SIDE EFFECTS

There are no known side effects. However, because Multi-Gyn FloraPlus has an optimal acidity (low pH) for the vagina and sperm is alkaline (high pH), it may reduce the activity of sperm. Therefore we recommend not to use Multi-Gyn FloraPlus during ovulation period when you wish to become pregnant.

## INTERFERENCE WITH MEDICATION

Multi-Gyn FloraPlus has no known interactions with medication.

## PREGNANCY AND BREASTFEEDING

There are no known contraindications for the use of Multi-Gyn FloraPlus during pregnancy and while breastfeeding. Discuss the options with your healthcare professional.

## USE WITH CONDOMS AND PESSARIES

The effect of Multi-Gyn FloraPlus on a pessary and condoms is unknown.

## STORAGE

Keep Multi-Gyn FloraPlus at a temperature between 10°C and 25°C (room temperature). Store Multi-Gyn FloraPlus out of reach of children. Do not use Multi-Gyn FloraPlus after the expiry date.

## BACKGROUND INFORMATION

A healthy vagina hosts a balance of different bacteria - the vaginal flora. When the natural environment or pH of the vagina is disturbed harmful bacteria and microbes can thrive. Disruption of this balance is often noticed by a change in the vaginal discharge. Symptoms may involve itch, burning sensation and crumbly, white discharge.



### Distributed by:

Nice Pak Products  
120 Woodlands Drive  
Braeside, Victoria, 3195, Australia



### Manufactured by:

BioClin BV, Blaak 555,  
3011 GB Rotterdam, The Netherlands

